
Instructions For Use

embrace MINIMIZE®

By combining the restorative properties of silicone with the benefits of Stress-Shield™ Technology, this unique scar treatment softens, flattens, and fades old scar tissue to drastically improve a scar's appearance.

embrace MINIMIZE®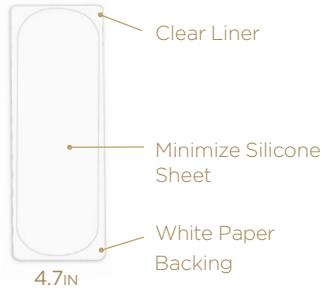

INTENDED USE

embrace MINIMIZE® is intended for use in the management of closed hyperproliferative (hypertrophic and keloid) scars.

CONTRAINDICATIONS

- Not for use on open incision lines, injuries or abraded skin.

WARNINGS & PRECAUTIONS

- DO NOT** use on scars if infection is suspected.
- Not a sterile device.
- For external (topical) use only.
- DO NOT** use if you have known adverse reactions to medical adhesives or skin diseases that would preclude the use of adhesive sheets.
- MINIMIZE is water-resistant. Short term exposure to water, (such as showering) is OK. DO NOT expose to water for prolonged periods of time.
- Wear time may be reduced in high mobility areas.
- DO NOT** pull quickly as skin irritation or tearing can occur.
- Keep out of reach of children.

- Any serious medical incident that occurs in relation to the embrace® device should be reported to Neodyne Biosciences and the Competent Authority of your member state.

POTENTIAL SIDE EFFECTS

Application of any surgical tape or adhesive sheet may strip the skin upon removal. Consult with your physician should any of the following occur:

- Skin irritation, redness, or pain
- Infection and/or fever
- Allergic reaction

BEFORE APPLYING MINIMIZE

- Position yourself in a way that ensures the area around the scar is as flat as possible, allowing for a natural range of motion once the sheet is applied
- Thoroughly clean and dry the scar and surrounding area


Applying MINIMIZE

- Peel open packet and remove product.** There are 3 silicone sheets on the tri-fold liner.

- Separate one sheet** by tearing along perforation.

- Peel off the white paper backing** from the clear liner.

- Hold the liner with one hand and partially peel** one corner of the minimize silicone sheet to expose the adhesive side.

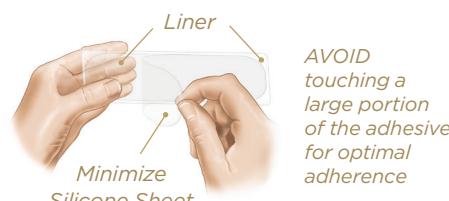- Place exposed sheet onto scar site with adhesive side on the skin.** Next press and hold sheet down to adhere to the skin.

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
- Gently peel** the remaining clear liner.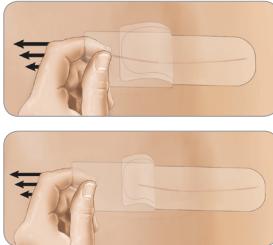

DO NOT stretch dressing, as this may cause irritation or blistering

- Pat and roll out** second half of the sheet so it firmly adheres to the skin, covering the entire scar site.

- Smooth out any air bubbles and apply pressure onto entire sheet for 60 seconds** to ensure adherence to the skin.

Smooth out with fingers to release air pockets

- Once applied, embrace MINIMIZE® can last up to 10 days, and is then discarded and replaced with a new sheet.

SCARS REQUIRING MULTIPLE DRESSINGS

For scars longer than a single dressing, dressings should overlap. Avoid gaps between dressings, and ensure scar is completely covered.

Removing MINIMIZE

- It is helpful to remove the sheet in the shower using warm water and soap.
- Gently peel the sheet back at 180°, rolling it back upon itself. Use your other hand to apply slight pressure on your skin as you remove the sheet.

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English | Anglais | Inglés

- Do not pull quickly as skin irritation or tearing can occur.
- If any adhesive remains, remove it with an oil based product such as baby oil or over-the-counter adhesive remover.

Helpful Hints

IMPORTANT THINGS TO REMEMBER

- Trim to size with rounded corners to avoid irritation.
- After initial application, embrace MINIMIZE® can be worn on average 10 days.
- Do not shower for 4 hours after application.
- Do not apply compression garments or binders for 1-2 hours after application.
- Minor redness is OK.
- Moisture or condensation under the sheet is OK (such as after showering).
- After application, a minor wrinkly or wavy appearance in the sheet is normal.

1.855.722.7879

Customer Service Telephone

Help@EmbraceScarTherapy.com
Customer Service E-mail

www.EmbraceScarTherapy.com
Product Website

youtube.com/EmbraceScarTherapy
Application/Removal Videos

www.EmbraceScarTherapy.com/how-to-use-minimize
Digital Instructions (this document)

Manufactured for: Neodyne Biosciences, Inc. | 47801 Fremont Blvd. | Fremont, CA 94538 | United States of America

Made in Mexico. Made with USA materials

CE REP Emergo Europe | Prinsesegracht 20, 2514 AP | The Hague, The Netherlands

embrace MINIMIZE® is a trademark of Neodyne Biosciences, Inc. © 2019 Neodyne Biosciences, Inc. All rights reserved. LS11667 Rev E. Patents: <https://www.embracecartherapy.com/citations-patents/>

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Fabricado para: Neodyne Biosciences, Inc. | 47801 Fremont Blvd. | Fremont, CA 94538 | Estados Unidos de América